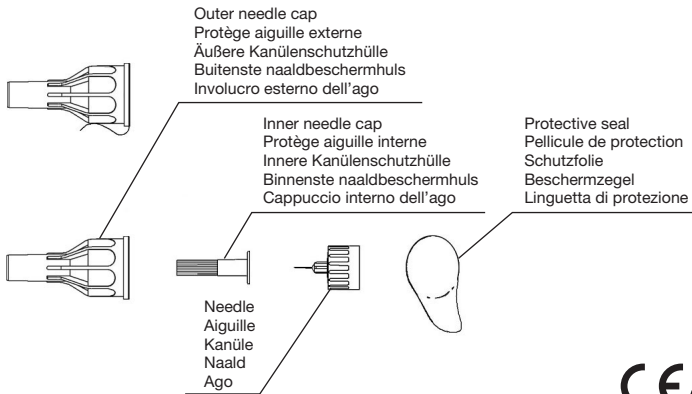
Product Drawing / Schémas du produit / Produktzeichnung / Schematische weergave / Descrizione dei componenti

	TERUMO CORPORATION		
	44-1, 2-chome, Hatagaya, Shibuya-ku, TOKYO 151-0072, JAPAN	MADE IN JAPAN	
	TERUMO EUROPE N.V.	Interleuvenlaan 40, 3001 LEUVEN, BELGIUM	
®: Registered Trademark	®: Marque enregistrée	®: Eingetragenes Warenzeichen	
®: Geregistreerd handelsmerk	®: Marchio Registrato	©TERUMO CORPORATION	May 2009 09E25

DEUTSCH

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ENGLISH

Intended Use:

This product is intended for use with a pen injector device for the subcutaneous injection of drugs including insulin and human growth hormone.

Please read carefully before use.

Caution:

- The device is intended for single use only. Do not reuse. Do not resterilize. Do not reprocess as reprocessing may compromise the biocompatibility and functional integrity of the product.
- Always use a new needle for each injection.
- Use following the instructions of physician or medical professional.
- Do not use if protective seal is opened or damaged.
- Do not use expired needles.
- Use immediately after protective seal is opened.
- Do not use if needle is bent or damaged.
- Do not bend the needle.
- Do not touch the needle directly.
- Dispose of safely after single use to avoid risk of infection.
- Be careful about needle stick injury.

Instructions for Use:

- See your Drug Delivery System Instruction Manual for information on assembly and injection.

[How to attach needle to the pen-injector]

- Peel off the protective seal.
- Attach the needle straight to a pen-injector, and screw it clockwise until it stops.
- Remove the outer needle cap and inner needle cap straight.

[How to detach needle from the pen-injector]

- Only in case of self-injection, carefully replace the outer needle cap straight. Employees in healthcare facilities need to follow special instructions from the facility.
- Hold the pen-injector firmly while you screw the needle counterclockwise and remove it from the pen-injector.
- Dispose of safely to a special container to prevent accidental needle stick injury.

Caution in Use:

- When removing the inner needle cap, remove it straight to avoid touching the needle.
- Disinfect the puncture site before each injection.
- Take care to place injection at least 2-3 cm apart from the prior injection site or to inject at contra lateral sites.
- Change injection site regularly following the instructions of physician or medical professional.
- Check the flow of the drug through the needle before each injection.
- Don't change the angle of the pen after penetrating the skin in order to avoid breaking of the needle. In case the needle broke off and remains in the body, please contact immediately a doctor.
- Attach a new needle before each injection and remove it immediately after the injection.
- Do not leave a needle attached to a pen-injector. Otherwise there is a risk of accidental needle stick injuries, air entering the cartridge and the needle being clogged with the drug.

Storage conditions:

- Do not store at extreme temperatures and humidity.
- Keep away from children.

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